

Instructions for proper use

Hoof Boot SG / SG-L



1

The fetlock strap must be attached once before first use using the short Velcro inside the slot of the bale shell.



2

Loosen the fastening strap and attach the Velcro on the side of the central bar.



3

Slide the hoof boot over the hoof from the tip. The buckle of the fastening strap must be visible on the outside.



4

With a light tap on the front of the hoof boot, ensure that the hoof is fully forward in the boot. Make sure the boot is straight. For horses with long feathering, pull out the hair.

IMPORTANT NOTES

Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH
Gass 12
CH-5242 Lupfig
Phone +41 (0)52 763 43 43
info@swiss-galoppers.com

For more information, visit
www.swiss-galoppers.com



Instructions for proper use

Hoof Boot SG / SG-L



5

Now place the hoof down and check, by lifting the bale shell, that the boot fits correctly.



6

IMPORTANT! Thread the fetlock strap through before tightening the fastening strap. Press the heel shell back slightly to do this.



7

Tighten the fastening strap and close the boot at the front so that the fastening strap fits neatly all around. In the final position, the metal buckle is directly in front of the side strap guide.



8

Now close the fetlock strap. The fetlock strap should fit snugly but not be too tight (one finger wide).

IMPORTANT NOTES

Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH
Gass 12
CH-5242 Lupfig
Phone +41 (0)52 763 43 43
info@swiss-galoppers.com

For more information, visit
www.swiss-galoppers.com



Important Notes

Hoof Boot SG-L



The L-Version is used when the standard boot does not optimally cover the length and/or height of the hoof, causing the heel shell to sit too low or not vertically aligned.

Angles & Fit



Ideal angle



Angle too flat


Ballenschale



Ideal height of the heel shell.



Heel shell sits too high

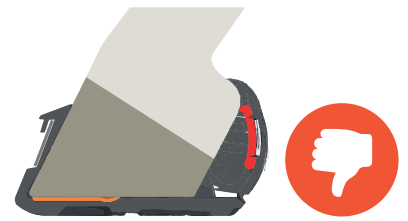

Heel shell too deep

IMPORTANT NOTES

Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH
Gass 12
CH-5242 Lupfig
Phone +41 (0)52 763 43 43
info@swiss-galoppers.com


For more information, visit
www.swiss-galoppers.com



Replacing the Extension

Hoof Boot SG-L

Attach the Retaining Bracket

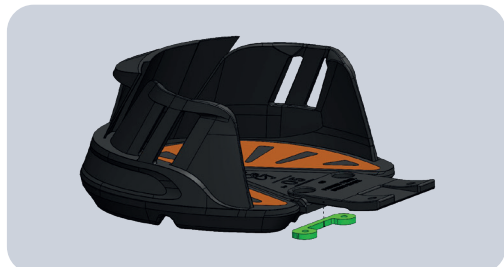

1

Place the retaining bracket with the thread into the pre-milled spot on the sole.



2

Align the retaining bracket and fasten it with two screws from the top (i.e., from the opposite side of the sole).



Note: The retaining bracket already has a thread – nuts are therefore not required.


Attach the Heel Shell



3

First, guide the screws through the backing plate.

Then insert the screws through the corresponding mounting holes of the heel shell.

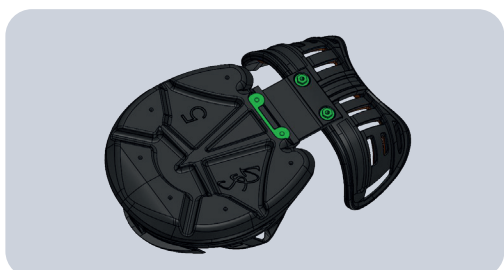

4

Screw the nuts onto the screws from the back and tighten them so that the heel shell is securely fixed to the extension.



Important maintenance note: Regularly check the screws and tighten them if necessary to ensure a safe and stable connection over time.

IMPORTANT NOTES

Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH
Gass 12
CH-5242 Lupfig
Phone +41 (0)52 763 43 43
info@swiss-galoppers.com

For more information, visit
www.swiss-galoppers.com

