

Instructions for proper use Hoof Boot SG / SG-L



The fetlock strap must be attached once before first use using the short Velcro inside the slot of the bale shell.



Loosen the fastening strap and attach the Velcro on the side of the central bar.



Slide the hoof boot over the hoof from the tip. The buckle of the fastening strap must be visible on the outside.



With a light tap on the front of the hoof boot, ensure that the hoof is fully forward in the boot. Make sure the boot is straight. For horses with long feathering, pull out the hair.



Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH Gass 12 CH-5242 Lupfig Phone +41 (0)52 763 43 43 info@swiss-galoppers.com

For more information, visit www.swiss-galoppers.com





Instructions for proper use Hoof Boot SG / SG-L



Now place the hoof down and check, by lifting the bale shell, that the boot fits correctly.



IMPORTANT! Thread the fetlock strap through before tightening the fastening strap. Press the heel shell back slightly to do this.



7 Tighten the fastening strap and close the boot at the front so that the fastening strap fits neatly all around. In the final position, the metal buckle is directly in front of the side strap guide.



Now close the fetlock strap. The fetlock strap should fit snugly but not be too tight (one finger wide).

i IMPORTANT NOTES

Do not use hoof grease or similar greasy products such as oils or creams! These reduce the boot's optimal grip!

To extend the lifespan of your Swiss Galoppers, regularly clean the fastening and fetlock straps by rinsing with water and always close the Velcro fasteners when not in use.

Swiss Galoppers GmbH Gass 12 CH-5242 Lupfig Phone +41 (0)52 763 43 43 info@swiss-galoppers.com

For more information, visit www.swiss-galoppers.com

